



ESC2000 Illuminated ETDRS Cabinet


Operation

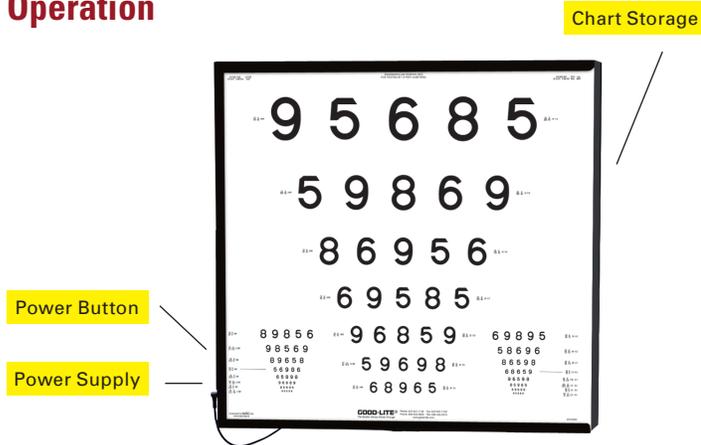


ESC2000 with optional stand

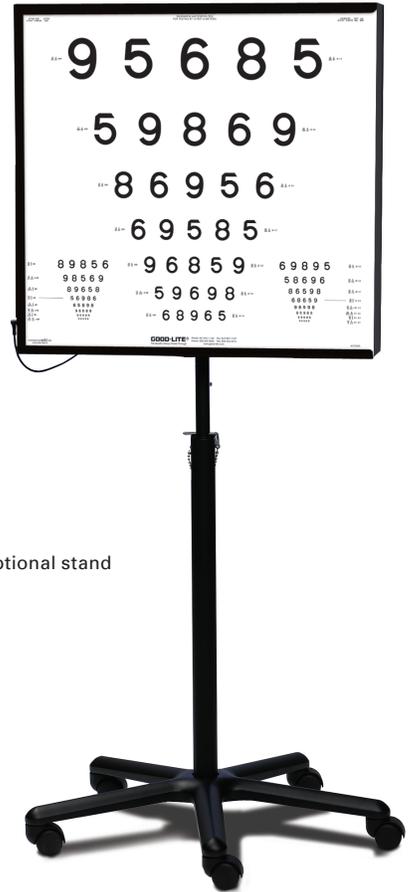

1. Plug power supply into ESC2000

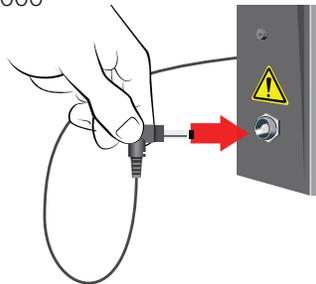

2. Plug power supply into electrical outlet

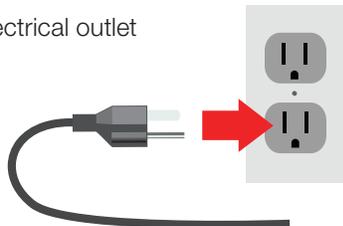

3. Press On/Off button to turn ESC2000 on.
Press again to turn illuminator off.



Calibrating ESC2000 Light Level

Note: To properly adjust light level, a light meter will need to be used. This unit uses uncalibrated light

Enter Calibrate Mode:

1. Unplug power supply from ESC2000
2. Press and hold switch while plugging power supply back into the ESC2000.

Unit will turn ON and “blink” once. Release button. The ESC2000 is now in Calibration Mode.

Quick (< 1sec) presses of button will step increase voltage output which will increase the brightness of the LED lights.

To decrease voltage and LED brightness, go to step 1 and repeat.

Exit Calibrate Mode and Save Calibration:

1. Press and hold button until unit turns off (approx. 5 seconds).

ESC2000 is now in Operate Mode and ready to use.